


Suggested Electrode Placement Chart

The sticky electrode pads need to be placed in the correct position for maximum pain relief. It is preferable to obtain assistance from either your birthing partner or medical adviser when positioning the pads. Please read the following instructions carefully:

1. Ensure unit remains switched off. Carefully peel the pads from the plastic backing using your forefinger and thumb. Do not pull on the lead wires.

2. Position the top two pads (connected to the left socket CH1) approximately 4cm either side of the spine and 4-5cm above the imaginary navel line as seen in the diagram.

3. Position the bottom two pads (connected to the right socket CH2) approximately 4cm either side of the spine and 4-5cm below the imaginary navel line as seen in the diagram.

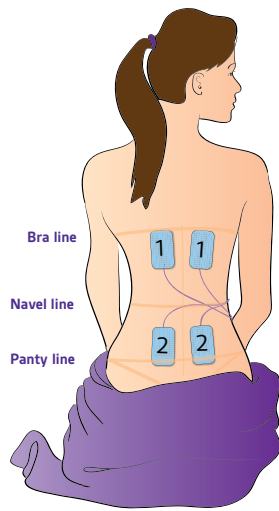

Reference: Cluett E. (1994) SRN RM ADM PGCEA. Analgesia in Labour: A review of the TENS method. Professional Care of Mother and Child. Mar;4 (2): 50-2.

7

Frequently Asked Questions

Q When shall I start using my Elle TENS?

A Use as early as you can after the onset of labour to allow time for your body's pain-killing chemicals to rise.

Q Can I try the Elle TENS before going into labour?

A Yes, place 2 pads attached to one leadwire on your forearm to feel the different pulse sensations following the instructions on page 6.

Q Can I combine the Elle TENS with other medication?

A Yes, you can still use analgesics such as gas and air (entinox) or pethidine.

Q Can the Elle TENS be used in hospital?

A Yes. However, a midwife may ask you to switch it off temporarily.

Q Can I use the Elle TENS if I am considering a water birth?

A Yes, but not in the water itself.

10

Using Your Elle TENS

a) You are now ready to start using your Elle TENS. Switch the unit on by pressing the **Ⓛ** on/off button. Ensure that the 'boost' button located at the top right of the unit is **NOT** pressed in. The Elle TENS will always start up in BURST 1 mode.

b) Begin by pressing the **+** CH1+ key. Each time you press this key, a small 'half-moon' shape will appear on the screen increasing in number and size as the intensity is increased. You will begin to feel a pulsating sensation which will burst in time with the small arrow flashing on the screen. **Ensure that you increase the intensity slowly.**

c) Repeat the previous step with the **+** CH2+ key. You will now feel a sensation in both sets of pads.

d) **BURST 1 (use between contractions):** This is the first mode that you will use. It feels like a bursting, tingling sensation i.e. the pulses are off and on periodically. This mode will help to promote those all-important natural pain-killing chemicals known as 'endorphins' and 'encephalins'.

e) **BOOST 1 (use during contractions):** When experiencing a contraction, simply press the 'boost' button located at the top right of the unit. The Elle TENS will then switch automatically into the BOOST 1 mode for that extra surge of power required to combat the pain during contractions. This will feel like a continuous sensation.

f) Once the contraction has passed, simply release the 'boost' button. The Elle TENS will return to the original BURST 1 mode. Continue this cycle during your early stages of labour.




8

Useful Information

- Continue stimulation for as long as necessary, unless you experience discomfort.
- If you stimulate for a number of hours ensure the pads do not become dry. If this occurs, switch off the unit, disconnect the electrodes, and apply a small drop of water to them.
- The pads can be used several times. Replace the pads when they stop sticking to the skin. Replacement pads are available from www.babycaretens.com
- Always ensure that the unit is switched off before you remove all the leads and pads. Do not pull the wires. ALWAYS remove the jack plug from its socket by holding the plug between the forefinger and thumb and pulling gently.
- The Elle TENS can also be used as a standard TENS unit for relief of numerous conditions, including after pains and post operative pains. It is worth bearing this in mind if you have a caesarean section. Please seek medical advice first.

Post-Natal Pad Placement Diagrams

KEY:
●● = first pair of electrodes
○○ = second pair of electrodes
N.B. Electrodes supplied with this unit are all the same colour - the black and white squares show suggested electrode placement only.

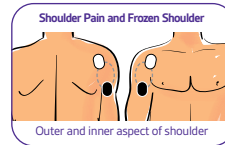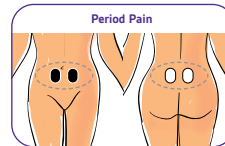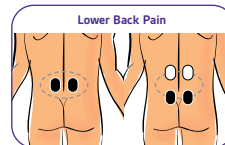

11

The Mode Button

There are two modes available on the Elle TENS, each with a Burst and Boost function.

Mode 1 (Burst 1 and Boost 1), which has been described above, is primarily used for the earlier stages of labour.

Mode 2 (Burst 2 and Boost 2) can be used when your labour advances and your contractions become more frequent.

Press the **Mode** button to switch between modes. You will see from the User Display, that the number in the small circle next to the BURST or BOOST will change from 1 to 2 and vice-versa.

NB. Both modes are interchangeable and can be used according to your preference, not necessarily for the stage of labour you are in. Mode 2 can be operated in the same way as mode 1.

The Opti-max Key

This provides extra versatility to your Elle TENS. Increasing or decreasing the Opti-max key will allow you to either heighten or lessen the level of pulse power transmitted to the electrode pads.

Pressing the **OM** key either + or - will increase or decrease Opti-max sensation, this is extremely useful for the final stages of labour when you need an extra surge of power.

There are 5 Opti-max levels indicated by 5 dots, enlarging in size as the Opti-max power is increased. The unit will always start up at Opti-max 3, which is exactly half way between the highest and lowest setting.

NB. The Opti-max key can be used according to your preference and not necessarily for the stage of labour you are in.

9

Elle TENS Technical Data

Display	TN type LCD		
Operation Mode	4 treatment modes: Burst 1, Burst 2, Boost 1, Boost 2		
	Pulse (Hz) Frequency	Pulse Width (µs)	Description
Burst 1	16	150*	1 Burst/sec and 8 pulses per burst
Burst 2	32	150*	2 Burst/sec and 8 pulses per burst
Boost 1	80	150*	Constant
Boost 2	100	150*	Constant
	- Output Voltage: 0 – 55V zero to peak at 5000 load - Output Waveform: Symmetrical biphasic rectangular		
Automatic Power Off	5 Minutes		
Output Channel	Two Channel		
Output Voltage	0 – 55V (zero-to-peak) across a 5000 load; adjustable in 15 steps		
Output Intensity	0 – 110 mA (max.) into a 5000 load		
Output Waveform	Symmetrical bi-phasic rectangle		
Output Jacks	CE Touch-proof jacks		
Battery	2 x 1.5V AA Battery		
Battery Low Detect	2.2V ±0.2V		
Tolerances	All output parameters are subjected to a ±20% tolerance unless other specified.		
Weight	75g		
Size	60 x 120 x 35mm		

* Default pulse width after mode changed (pulse width is **NOT** changed when switched between BURST and BOOST mode) 5 selectable pulse widths in all modes: 50, 100, 150, 200, 250.

Classification This unit is fully compliant with EEC Directive 93/42/EEC, classified as internally powered equipment type BF. It is intended for continuous operation.

12

Scan here!

Grab your download

A Women's Guide to Drug Free Pain Relief



A Women's Guide to Drug Free Pain Relief

Instructions for use

ELLE TENS

Babycare TENS

Guarantee

Your ElleTENS machine is guaranteed for a period of 1 year against manufacturer's defects. Please record purchase details in the space provided below for your record purposes. The guarantee does not include leads, electrodes or battery. Applies to purchases only.

Date of purchase:

Invoice number:

Babycare TENS

Body Clock Health Care Ltd
108 George Lane, South Woodford,
London E18 1AD

Tel (+44) (0)20 8532 9595
www.babycaretens.com

e-mail customercare@babycaretens.com

MedEnvoy Global B.V.
Prinses Margrietplantsoen 33 - Suite 123 2595 AM
The Hague The Netherlands

Babycare TENS is part of the
Body Clock group of companies

©1989-2025 Body Clock Health Care Ltd.
All rights reserved.

1639
Feb 2026 (14)
Model: D-BC04T
ELLE-IFU-1

recycle
This manual is recyclable

Elle TENS

Quick Reference Guide

If you are in labour, please use the quick reference guide set out below. It is however recommended that you read the section entitled "Setting Up Your Elle TENS" found on page 5 to obtain the best performance from your unit.

Before using the Elle TENS, please read the contraindications and warnings on pages 1 and 2.

- Ensure the Elle TENS unit is switched off.
- Insert 2 x AA batteries in the compartment at the back of the unit (see figs 1 and 2, page 6).
- Insert the leadwire plugs into the top of the unit (see fig 3, page 6).
- Insert the pins at the other end of the leadwires into the sockets attached to the electrode pads. Do not remove the electrodes from their backing at this stage (see fig 4, page 6).
- Position the electrode pads on the body as indicated on page. A pair of electrodes must be connected to each lead wire in order for you to feel any sensation. The electrode pads must be peeled away from their backing and placed directly on your body for you to feel any sensation.
- Make sure that the 'boost' button at the side of the unit is **NOT** pressed in. This ensures that the unit will start in the BURST 1 mode (see page 3).
- Press the (on/off) to turn the unit on.
- Now push the (CH1+) key until you feel a pulsating sensation.
- Repeat the same with the (CH2+) key.
- At the onset of a contraction press the 'boost' button for the additional surge of power needed to combat the pain.
- Release the 'boost' button once the contraction has passed and continue in the BURST 1 mode as before.

Controls For The Elle TENS

The Elle TENS is very simple to use. However, it is advisable to read these instructions to obtain the most out of the unit. Please ensure that all of the precautions have been read thoroughly. See page 2.



Contents

Elle TENS Quick Reference Guide

How Will The Elle TENS Help Your Pain?.....	1
Introduction To The Elle TENS	1
General Precautions.....	2
Controls For The Elle TENS	3
Display Symbols and their Meanings.....	4
Setting Up Your Elle TENS	5
Useful Diagrams	6
Suggested Electrode Placement Chart.....	7
Using Your Elle TENS	8
Frequently Asked Questions	10
Troubleshooting Questions.....	10
Useful Information.....	11
Post natal pad placement diagrams.....	11
Technical Specifications.....	12

Explanation of Symbols on Unit


Equipment providing a particular degree of protection against electric shock particularly regarding allowable leakage currents having an F type (floating) applied part.

Warning - refer to pages 1-2 of these instructions.

EU Representative.

This unit needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the ACCOMPANYING DOCUMENT.

Display Symbols And Their Meanings

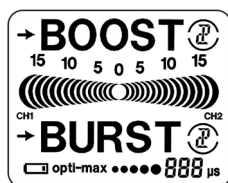

Power Bar: This shows the output from 0 to 15 for both left and right channels. Each segment represents approximately 7% of the total output.

BOOST: Pressing the 'Boost' button at the top right of the unit activates the Boost mode. This function is used during contractions. The small arrow to the left of the word Boost will remain on, indicating the unit is in a continuous mode. There are two Boost modes available indicated by a 1 or 2 in a small circle next to the word BOOST. You can switch between them by pressing the Mode key.

Please keep device away from sprays of water or rain.

IP22 The first number 2: Protected against access to hazardous parts with a finger, and the jointed test finger of 12 mmΦ, 80 mm length, shall have adequate clearance from hazardous parts, and protected against solid foreign objects of 12.5 mmΦ and greater. The second number 2: Protected against vertically falling water drops when enclosure tilted up to 15°. Vertically falling drops shall have no harmful effects when the enclosure is tilted at any angle up to 15° on either side of the vertical.

Denotes a product which must be disposed of safely.

LOT Batch code identifying the production run of the unit.

UDI Unique device identifier - Human and machine-readable unique identifier of the individual unit indicating its product code, batch number and date of manufacture.

How Will The Elle TENS Help Your Pain?

TENS (Transcutaneous Electrical Nerve Stimulation) works by passing mild electrical impulses through the skin, via electrode pads, into the nerve fibres which lie below. The TENS impulses help your body produce its own pain killing chemicals, such as 'endorphins' and 'encephalins'.

The level of pain relief obtained varies from person to person. Some find that the Elle TENS provides all the pain relief required during the whole of labour, whereas others find that other analgesics are also needed during the later stages of labour.

Introduction To The Elle TENS

The Elle TENS uses a state-of-the-art, efficient method of drug-free pain relief with no known side effects. In order to obtain maximum benefit, it is advised that you seek guidance from your medical adviser as well as reading this instruction guide.

Contraindications

- Do NOT use if you have a demand-type pacemaker or implanted electrical device
- Do NOT use on infants or people not capable of expressing their intentions.

Warnings

- Do NOT use before 37 weeks of pregnancy unless advised by a medical professional.
- Do NOT place the electrodes over your abdomen when pregnant.
- Check with your healthcare professional before use if you suffer from a heart condition, epilepsy, undiagnosed pain, varicose veins, recent scarring, have any metal implants, or any doubts whatsoever.
- Do NOT use if you experience an allergic reaction to the electrodes.
- Do NOT place electrodes/probes across or through the head, near the eyes, covering the mouth, on the front or sides of the neck (especially the carotid sinus), on the chest and upper back crossing over the heart, directly over your heart or an area of broken, inflamed, infected or numb skin unless under medical supervision.
- Do NOT use in the presence of tuberculosis, malignant tumours, extreme blood pressure, high fever or acute inflammatory disease, unless under medical supervision.
- Do NOT immerse this device in any liquid. Keep it away from sources of excessive heat and flammable gas.
- Do NOT drop this unit onto a hard surface.
- Do NOT attempt to dismantle, open up or repair this device. Instead, return it to the manufacturer.
- Do NOT modify this equipment device in any way or interconnect it with any other equipment not described in these instructions for use.
- Do NOT use while driving, operating potentially dangerous machinery or using a microwave.
- Do NOT use in close proximity (e.g. 1m) to shortwave or microwave therapy equipment to avoid instability in the stimulator output.
- Be aware of the risk of strangulation and asphyxiation from lead wires. Do NOT use this device while asleep.
- Do NOT use battery or power sources other than those specified in these instructions.
- Electronic monitoring equipment (such as ECG monitors and ECG alarms) may not operate properly when TENS is in use.
- Do NOT use if you experience an allergic reaction to the electrodes. Mild temporary skin irritation can occur following long-term application. STOP using the device. Seek medical assistance.
- Using this device with high-frequency surgical equipment may cause burns. Users should seek medical advice when use of this equipment is required.
- Keep out of the reach of children and babies. They may be at risk of strangulation by entanglement in lead wires or choking by swallowing small parts or accessories.
- It is unsafe to use accessories, detachable parts and materials not described in these instructions for use. Only use specified accessories.
- Keep this device away from lint and dust. Long term exposure may affect the connections.
- Do NOT use if the device or accompanying accessories are opened or damaged.
- A mild shock may be caused if one electrode is accidentally removed. In the event of this happening the unit should be turned off immediately.
- Never use TENS to mask undiagnosed pain since this could require urgent treatment.

Contraindications

- Read all instructions before use.
- This unit needs special precautions regarding EMC and needs to be used according to the EMC information provided in the accompanying EMC document.
- Turn device off if an electrode detaches unexpectedly since this may cause a mild shock.
- Electrodes should only be positioned as recommended for childbirth (see page 7), unless otherwise stated by your midwife or medical adviser.
- Continue stimulation for as long as needed, unless you experience discomfort.
- If you stimulate for a number of hours ensure the pads do not become dry. If this occurs, switch off the unit, disconnect the electrodes, and apply a small drop of water to them.
- If damaged, do NOT use. Return to supplier.
- The device shall not be serviced or maintained while in use.
- Remove the batteries when not in use.
- Do NOT use if you experience an allergic reaction to the electrodes.

Setting Up Your Elle TENS

Check the following contents:

- 1 x Elle TENS unit
- 1 x Pack of 4 self adhesive electrodes (40mm x 100mm)
- 2 x Leadwires
- 2 x AA batteries
- 1 x Carrying pouch
- 1 x Easy-release neck cord
- 1 x Instruction manual
- 1 x EMC documentation

The Assembly Stage

- Press and rotate the belt clip at the back of the Elle TENS in either direction to reveal the battery compartment (see fig 1, page 6).
- Release the battery compartment cover and insert 2 x AA batteries (see fig 2, page 6), ensuring the positive (+) and negative (-) terminals are correctly positioned as marked in the battery compartment. Replace the cover. **Ensure the unit is switched off.**
- Take two of the purple leadwires and insert each of

the black jack plugs (at one end of each leadwire) into the sockets at the top of the Elle TENS unit (see fig 3, page 6).

- Now insert the pins (at the other end of the leadwires) into the sockets attached to the sticky electrodes. Each leadwire has two pins, ensure each pin is attached to an electrode. Do not remove the electrodes from the backing at this stage (see fig 4, page 6).

NB. Each leadwire has a red and black pin. It makes NO difference whatsoever which electrode they are attached to!

- If you wish, attach the easy-release neck cord which enables you to hang the Elle TENS from your neck whilst in labour (see fig 5, page 6). Now place your electrodes (see page 7).

NB. The unit should be wiped clean periodically using medically approved anti-bacterial wipes or a very slightly dampened cloth. A mild soap may be used but DO NOT apply solvents.

Useful Diagrams



Figure 1

Inserting the battery

Press and rotate the belt clip to reveal the battery compartment.



Figure 2

Release the battery compartment and insert the batteries.



Figure 3

Connecting the leads

Insert both black jack plug into the sockets.



Figure 4

Connecting the Electrode Pads

Insert a red and black pin (at the end of each lead wire) into each pair of electrodes.



Figure 5

Using the Neck Cord for mobility during labour

Thread the loop of the neck cord through the space at the base of the unit. Slip the other end of the neck cord through the loop and pull into position.